SafeTest 60 QUICK START GUIDE

Warnings and Cautions


- Warning of electrical danger!**
Indicates instructions must be followed to avoid danger to persons.
 - Important, follow the documentation!**
This symbol indicates that the operating instructions must be adhered to in order to avoid danger.
 - Warning**
During many of the leakage tests the PROTECTIVE EARTH/GROUND CONDUCTOR to the appliance under test is interrupted. For this reason additional means of protection are required in order to comply with the relevant safety standards.
 - Safety**
Do not connect any probe combination to the maximum permitted voltage of 30VAC/DC with respect to earth/ground potential when performing non-power tests. Danger of electric shock.
- It must be assumed that safe operation is no longer possible:
- If the instrument or leads show any sign of damage
 - If the instrument does not function
 - After long periods of storage under adverse environmental conditions

rigelmedical.com

Introduction

Welcome to your new **SafeTest 60** hand-held Electrical Safety Analyzer. This Quick Start Guide tells you how to set up your **SafeTest 60** and make use of its key features.

1 Getting to know your SafeTest 60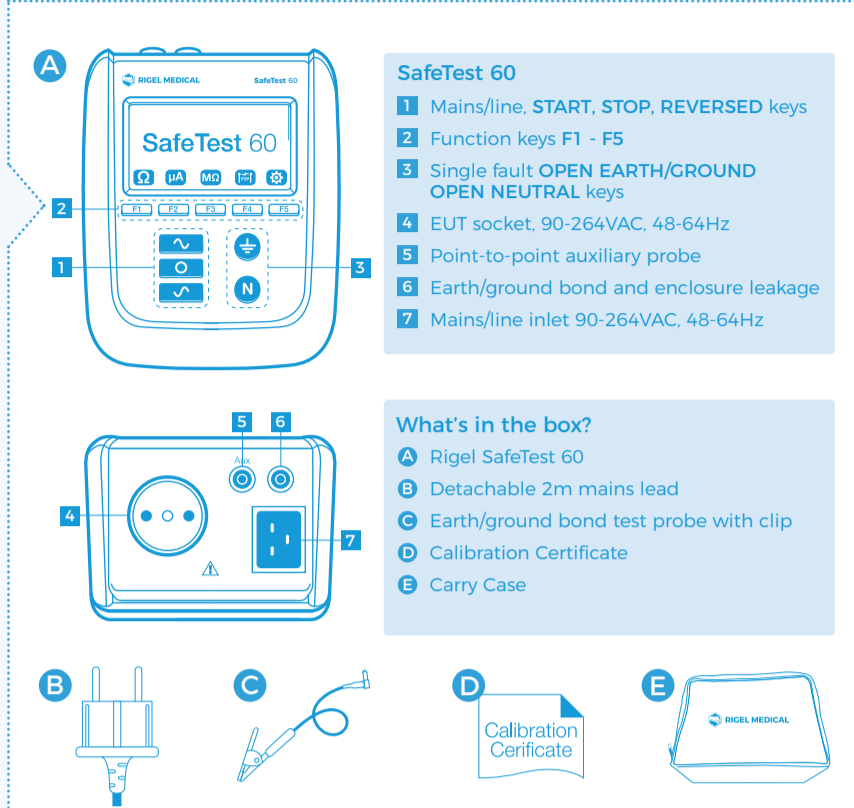

NOTICE

If any items are missing on receipt of your new unit, please contact your equipment supplier immediately.

2 Getting started

In this section we will identify some of the basic functions of the Rigel **SafeTest 60** and how to get you started.

Switch on

To switch on the Rigel **SafeTest 60**, please insert the power cord to the power inlet. The **SafeTest 60** will automatically power up.


3 Perform electrical safety tests

From the HOME screen, select the required test:

3a Settings menu

Press from the HOME screen to enter SETTINGS menu: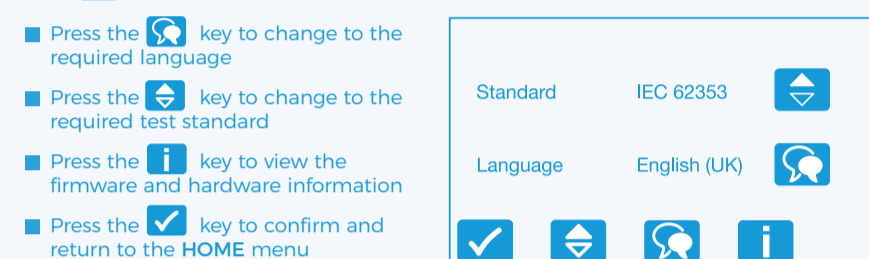

Need help?

This Quick Start Guide is designed to be used in conjunction with the full **SafeTest 60** instructional manual, available as a download from rigelmedical.com/safetest60

For technical issues please visit rigelmedical.com/support

For further assistance please contact your Rigel Medical supplier or use the following contact details to speak to a member of the Rigel Medical team:

Sales and Delivery enquiries
Email: sales@rigelmedical.com
Tel: +44 (0) 191 587 8730 Fax: +44 (0) 191 586 0227
USA office: +1 813 886 2775

Technical enquiries
Email: support@rigelmedical.com
Tel: +44 (0) 191 587 8701 USA office: +1 813 886 2775

Service, Calibration and Repair
Email: service@rigelmedical.com
Tel: +44 (0) 191 587 8739 Fax: +44 (0) 191 518 4666
USA office: +1 813 886 2775

Remember to activate your 2 year warranty* by registering your Rigel **SafeTest 60** with Rigel Medical

Go to rigelmedical.com/register
*Terms and Conditions apply. See website for details.

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