



**RIGEL MEDICAL**  
GMC-INSTRUMENTS GROUP

## What test fluids can I use for the Multi-Flo?

The following test fluids are acceptable for use with the Multi-Flo:

**De-ionised water**

**Distilled water**

**De-ionised water/distilled water with approximately 0.1% Micro-90 cleaning solution**

A 0.1% Micro-90 cleaning solution is the preferred test fluid as this will prevent the growth of organic material within the fluid path and also act as a wetting agent. This will reduce the likelihood of small air bubbles being trapped in the fluid path which could influence the measurement accuracy

Note: Do not use saline or dextrose solution as a test fluid. IV administration sets which may have been used with saline or dextrose solution should not be used with the Multi-Flo unless thoroughly cleaned beforehand. If saline or dextrose solution is accidentally used with the Multi-Flo, the unit should be cleaned immediately using the process described in section 5

If you require more help, please contact us at <https://www.seaward.com/gb/enquire/>.