



**RIGEL MEDICAL**

GMC-INSTRUMENTS GROUP

## How do I download test sequences to the Uni-Therm?

The file download has the most common test sequences available for the Uni-Therm. See below for the full list.

How to download the test sequences:

1. Download the file to an empty memory stick.
2. Connect the memory stick to the Uni-Therm USB slot.
3. Go to Menu-Data-Transfer Data- select Import test sequences using the F2 button- Select Memory stick in the port area using F3, then select F4 to start the transfer.
4. To make sure you have successfully downloaded the test sequences- from the home page press Menu- Set Up - Test Sequences. You will see the list of test sequences available.

**Please note that we are not responsible for the use of these test sequences and they should be used in conjunction with the OEM service procedure.**

Bovie Aaron 950

Bovie IDS Series

Conmed CB200 Beamer plus Argon

Conmed CE6000 Beamer

Conmed Hyfrecator 2000

Conmed System 5000  
Conmed System 7550  
Covidien Valleylab Force 2  
Covidien Valleylab Force FX  
Covidien Valleylab FT10  
ERBE 300 VA  
ERBE ICC 200  
ERBE ICC 300  
ERBE ICC 350  
ERBE VIO 200S  
ERBE VIO 300  
ERBE VIO 300D  
Medtronic AEX  
Medtronic Covidien ValleyLab Force Triad  
Medtronic Ligasure  
MEGADYNE MEGAPOWER  
Olympus ESG-100  
Olympus ESG-400  
Olympus PSD-20  
Olympus PSD-30  
Olympus PSD-60  
Olympus UES -30  
Olympus UES-40  
Valley Force 1  
ValleyLab FX  
Valleylab FX 8C  
ValleyLab FX-C  
Vesalius  
WOLF

If you require more help, please contact us at  
<https://www.seaward.com/gb/enquire/>.