



RIGEL MEDICAL

GMC-INSTRUMENTS GROUP

How can I fix the error code 26 in Med-eBase?

How can I fix the error code 26 in Med-ebase?

If SQL Server install is not Express edition please follow the instructions below:

1. Ensure the program is not running.
2. Open PC registry by selecting Start->Run then typing RegEdit and selecting OK.
3. Navigate to:

```
\\HKEY_LOCAL_MACHINE\SOFTWARE\Microsoft\Microsoft SQL  
Server\InstalledInstances
```

4. Make a note of the registry value at the above location then close the registry editor.
5. Open Windows Explorer by right-mouse clicking on Start and selecting Explore.
6. Navigate to the Med-eBase install directory, default is as follows unless changed during program installation:

```
C:\Program Files\Rigel\Med-eBase
```

7. Open Med-eBase.exe.config with Wordpad or Notepad.
8. Replace SQLExpress (highlighted in the figure 1) with the Installed Instance noted from the registry in step 3.
9. Save the file.

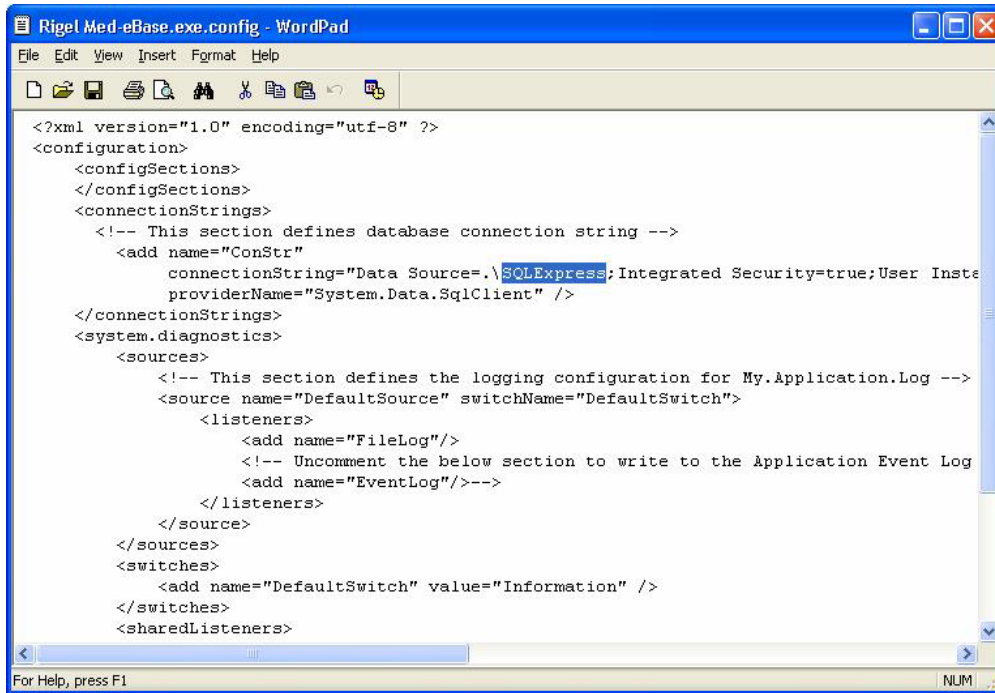


Figure 1

10. Restart the program – no error should be encountered when opening or creating a database.

If you require more help, please contact us at <https://www.seaward.com/gb/enquire/>.